**EN Dual-Valved Holding Chamber**

- Clean, Ready to Use • Easy Assembly • Printed with Soy Ink
- May be Flattened Between Uses • No Natural Latex

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
U.S. Patent No. 6,202,643; 6,550,473; 6,679,252 and International Patents.

LITEAIRE LOGO and THAYER MEDICAL LOGO are registered trademarks of Thayer Medical Corporation

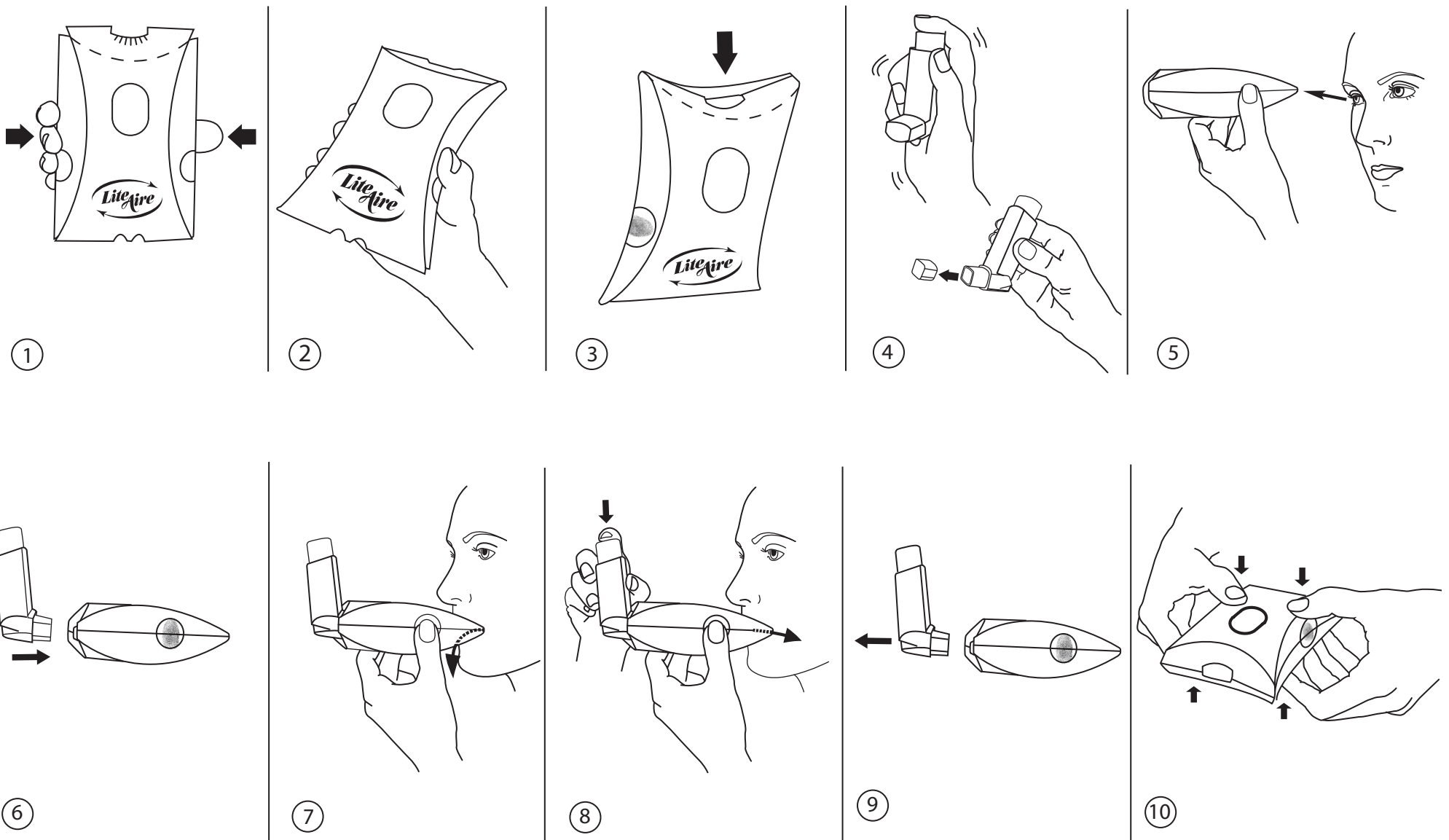© 2009 Thayer Medical Corporation

## INSTRUCTIONS FOR USE (single patient use only)

EN ENGLISH

### INTENDED USE

The LiteAire is a portable, pop-up dual-valved holding chamber designed to be used with virtually all metered dose inhalers (MDIs). The purpose of the LiteAire is to improve the delivery of the MDI medication to your lungs.

### HOW TO ASSEMBLE YOUR LITEAIRE

1. Place your fingers in the middle of the LiteAire along the long sides of the chamber. (Fig. 1)
2. Gently squeeze the device to make the LiteAire pop up fully. The side panels will snap into place. (Fig. 2 and Fig. 3)

### HOW TO USE YOUR LITEAIRE WITH YOUR MDI

1. Remove the cap from the inhaler mouthpiece. (Fig. 4)
2. Shake the inhaler prior to use as directed by the manufacturer. (Fig. 4)
3. Insert the inhaler canister mouthpiece firmly into the LiteAire inhaler port. (Fig. 6)
4. Firmly grasp your LiteAire with your thumb and fingers on either side at the point where you feel the paperboard inner wall. Gently squeeze the side panels to ensure the most effective treatment. (Fig. 7)
5. Close your lips firmly around the small holes on the mouthpiece of the LiteAire.
6. Exhale fully into the mouthpiece (Fig. 7). As you begin to inhale, press down on the metal canister to spray one puff of medication. Continue inhaling slowly and deeply. Hold your breath for 5 to 10 seconds. (Fig. 8)
7. If your physician has directed you to take more than one puff, repeat step 6.
8. Remove the inhaler and flatten LiteAire between uses. (Fig. 9 and Fig. 10)

### FOR BEST RESULTS

1. Hold side panels near the mouthpiece where you feel the paperboard inner wall.
2. For maximum results, start inhaling at the same time that you squeeze the MDI canister.
3. Inhale as slowly and deeply as possible through your mouth, to fill your lungs with the medication. Be sure to hold your breath for 5 to 10 seconds.
4. Each LiteAire can be used for multiple puffs. We recommend that you replace your LiteAire weekly or sooner if it is crushed, soggy, contaminated or discolored.

 **WARNING**  
Always look inside the mouthpiece and chamber before you use the LiteAire to be sure there are no small objects that could be inhaled (such as coins, lint, buttons, etc.).

Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner.

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