● Dual-Valved Holding Chamber

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


● Camera d'Espansione con Doppia

Valvola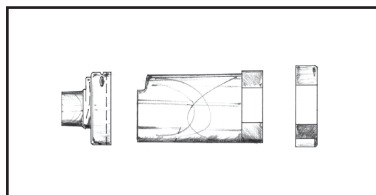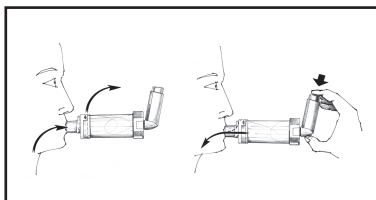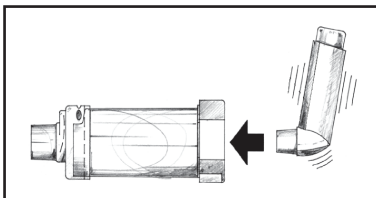

## INSTRUCTIONS FOR USE (single patient use only)

EN ENGLISH

### INTENDED USE

The PrimeAire is a portable, dual-valved holding chamber designed to be used with virtually all metered dose inhalers (MDIs). The purpose of the PrimeAire is to improve the delivery of the MDI medication to your lungs. The PrimeAire does this by reducing the amount of medication sprayed directly onto your throat, and by reducing the need to press the MDI canister at exactly the same time that you start breathing in.

### HOW TO USE YOUR PRIMEAIRE AND MDI

1. Remove the cap from the MDI mouthpiece.
2. Shake the MDI well, at least 4 or 5 times. (Fig. 1)
3. Insert the MDI mouthpiece firmly into the flexible oval opening in the MDI-adaptor end of the PrimeAire. The MDI canister must be pointed downward and the MDI mouthpiece should point straight toward the mouthpiece of the PrimeAire. (Fig. 1)
4. Close your lips firmly around the mouthpiece of the PrimeAire.
5. Exhale into the mouthpiece. Squeeze down on the MDI canister to spray one (and only one) puff of medication into the chamber. Inhale through the mouthpiece slowly (to avoid making the whistle sound), then hold your breath for 5 to 10 seconds. (Fig. 2)
6. If your physician has directed you to take more than one puff, repeat steps 2 through 5.

### FOR BEST RESULTS

- To get the maximum dose of medication, start inhaling at the same time that you squeeze the MDI canister.
- If the whistle sounds while you are inhaling, slow down your breathing to fill your lungs with the medication. Be sure to hold your breath for 5 to 10 seconds.
- To get all of the medication out of the chamber in one breath, inhale entirely through your mouth

### HOW TO CLEAN YOUR PRIMEAIRE

We recommend that you clean your PrimeAire once a week or more often if needed.

1. Remove the mouthpiece and the MDI-adaptor end from the chamber. (Fig. 3)
- NOTE: The mouthpiece end is not intended to be taken apart further.**
2. Gently clean all three parts in warm soapy water, using common dishwashing liquid. Thoroughly rinse all three parts in clear water then shake them to remove excess water.
  3. Allow the three parts to dry completely before putting them back together.

### FOR ADDITIONAL ASSISTANCE

Before using your PrimeAire for the first time, ask your physician, pharmacist, or other healthcare practitioner to show you how to use and clean it.

For patients having difficulty using the PrimeAire, a face mask may be used so that a caregiver may assist with the MDI treatment. When using a facemask, read the directions accompanying the face mask in addition to the directions for use of the PrimeAire.

### WARNING

Always look inside the mouthpiece and chamber before you use the PrimeAire to be sure there are no small objects that could be inhaled (such as coins, lint, buttons, etc.).

**Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner.**

Thayer Medical Corporation  
Tucson, Arizona 85714-1961 U.S.A.  
www.thayermedical.com

U.S. Patent No. 6,039,042

PRIMEAIRE LOGO and THAYER MEDICAL LOGO are a registered Trademarks of Thayer Medical Corporation © 2009 Thayer Medical Corporation

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