Valved Tee Adapter

22 mm Connections

For use with small volume nebulizers (SVNs)

INDICATIONS FOR USE

VALVED TEE adapters are single patient, disposable ventilator circuit accessories intended for use with small volume nebulizers (SVNs). These adapters are indicated for patients on a breathing circuit, for whom a physician or other licensed health care practitioner has prescribed nebulizer treatments. VALVED TEE adapters are intended for use by healthcare practitioners, in short and long term critical care environments. These adapters are for use with a 22 mm fitting.

DIRECTIONS FOR USE

1. Remove the VALVED TEE adapter from package.
2. Insert the VALVED TEE device into the inspiratory limb of the circuit.
3. Ensure a secure fit of the tubing with the VALVED TEE connectors.

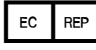
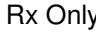
NOTE: When correctly installed, the VALVED TEE nebulizer port will be vertical and pointed downward during treatment.

4. Place the cap on the VALVED TEE adapter.
5. When ready for use, uncap the VALVED TEE. Press and twist the nebulizer into the TEE port. The valve is activated when the nebulizer is placed into the base of the TEE.
6. Once procedure is complete, twist and pull to remove the nebulizer.
7. Cap the VALVED TEE adapter.

NOTE: Discard the adapter when replacing the circuit according to facility guidelines, not to exceed two weeks.

WARNING: Do not clean and reuse or sterilize. Potential risks associated with cleaning and reusing, reprocessing and/or sterilization include: cross-contamination or infection resulting from inadequate cleaning, product degradation, and/or failure of the device to perform as intended.

Symbol Glossary

 Manufacturer	 Consult instructions for use
 Date of manufacture	 Authorized representative in the European Community
 Batch code/lot number	 Prescription only. U.S. Federal law restricts this device to sale by or on the order of a physician or properly licensed practitioner.
 CE Mark	 Caution. The symbol can also be used to mean Attention, see Instructions for Use.
 Catalog, reorder or reference number	

Item No.	Description
 1008	Valved Tee Adapter, 22 mm O.D. / O.D.
 1009	Valved Tee Adapter, 22 mm O.D. / I.D.

 Manufacturer Thayer Medical Corporation 4575 S. Palo Verde Rd. Tucson, AZ 85714-1961 USA www.thayermedical.com Made in USA	 0482	 MDSS GmbH Schiffgraben 41 30175 Hannover, Germany
---	--	---

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