1 Quake  
Made in USA

Consult Instructions for Use

**Manufacturer**  
Thayer Medical Corporation  
4575 S. Palo Verde Rd. Ste. 337  
Tucson, AZ 85714-1961 USA  
www.thayermedical.com  
G-1653\_PUB rev A

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0482

U.S. Patent NO. 5,658,221; 5,890,998; 5,899,832; 5,910,071 and International Patents

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician or other practitioner licensed to prescribe by law.

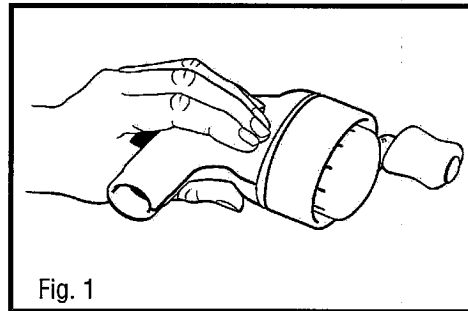
Fig. 1

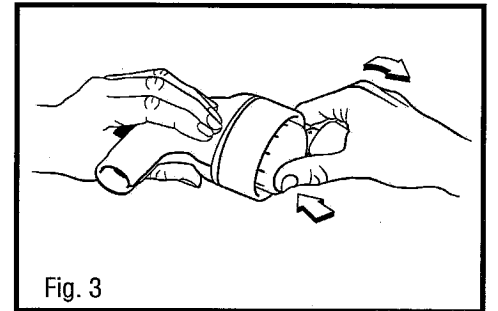
Fig. 3

Fig. 2

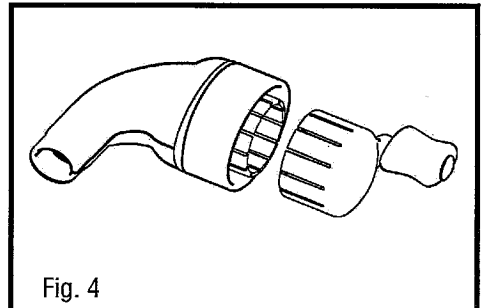
Fig. 4

ENGLISH

**INSTRUCTIONS FOR USE (single patient use only)**

**INTENDED USE**

The Quake is a portable handheld device designed to loosen secretions and assist in airway clearance of patients suffering from various respiratory diseases.

**CONTRAINDICATIONS<sup>1</sup>**

No absolute contraindications to the use of positive airway pressure have been reported; however, use in the following conditions should be carefully evaluated by a healthcare professional: untreated pneumothorax; hemodynamic instability; esophageal surgery; recent facial, oral, or skull trauma; active hemoptysis; inability to tolerate increased work of breathing; intracranial pressure (>20 mm Hg); acute sinusitis; epistaxis; nausea; known or suspected tympanic membrane rupture or other middle ear pathology.

**ADVERSE REACTIONS<sup>1</sup>**

Possible adverse reactions may include fatigue<sup>2</sup>; increased intracranial pressure; increased work of breathing that may lead to hypoventilation and hypercarbia; cardiovascular compromise; myocardial ischemia; decreased venous return; air swallowing with increased likelihood of vomiting and aspiration; claustrophobia; pulmonary barotraumas.

**HOW TO USE YOUR QUAKE**

1. You may use the device while in any position - sitting, standing or lying down - that is comfortable for you to perform your treatment.
2. Inhale deeply as instructed by your healthcare practitioner. Place your lips on the Quake mouthpiece and begin rotating the Quake's handle as you exhale at a slightly faster than normal rate.
3. Continue rotating the handle while inhaling and repeating the exhalation cycle, or proceed as directed by your healthcare practitioner.
4. Repeat Step 2 between three to six times. Suppress your desire to cough during these cycles.
5. Remove the Quake from your mouth and exhale **FORCEFULLY** to help loosen the secretions to aid airway clearance. This should help trigger a cough.
6. If Step 5 does not trigger a productive cough, attempt a "huff" cough to force the secretions from your airways.

**FOR BEST RESULTS**

1. The Quake is designed to allow you to change how quickly you exhale and rotate the handle to achieve the most effective clearance therapy. Alter both your inhalation and exhalation rate and the speed of the handle rotation to produce the vibratory level that best meets your needs.
2. Keep your cheeks rigid as you exhale so that the percussive effect is felt in your lungs rather than your mouth. This may take some practice; if you have any questions, please consult your healthcare practitioner.

**HOW TO CARE FOR YOUR QUAKE**

We recommend that you clean your Quake once a week or more often as needed.

**DISASSEMBLE:**

- a. To separate the gray barrel from the blue body, first notice that the blue piece is in the shape of an oval with the seams at the widest point.
- b. Line up the barrel handle with one of the seams.
- c. Hold the blue body firmly in one hand and tilt the handle toward the opposite seam while pulling the two pieces apart.

**CLEANING:**

After separating the gray outer barrel from the blue body, your Quake can be cleaned using one of the following methods:

- a. Hand wash: Gently clean both parts in warm soapy water, using common dishwashing liquid. Thoroughly rinse both parts in clean water and shake to remove excess water. Allow parts to air dry completely before reassembling.
- b. Dishwasher: Place both parts in the top rack of the dishwasher for a complete cycle.
- c. Boil: Boil both parts in water for ten minutes. Allow parts to air dry completely before reassembling.

**REASSEMBLE:**

Realign the gray barrel with the blue body and snap into place.

**NOTE**

Routinely inspect your Quake, particularly if dropped, to ensure it is functioning properly. Replace your Quake yearly, or sooner if needed.

**FOR ADDITIONAL ASSISTANCE**

Ask your physician, pharmacist or other healthcare practitioner to show you how to use and clean your Quake. For patients who have difficulty using the Quake, a face mask may be used with the assistance of a caregiver during treatment.

Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner.

**REFERENCES**

1. Adapted from the AARC Clinical Practice Guidelines: Use of Positive Airway Pressure Adjuncts to Bronchial Hygiene Therapy, Respiratory Care, vol. 38, #5, 1993.
2. Adapted from the AARC Clinical Practice Guidelines: Use of Incentive Spirometry, Respiratory Care, vol. 36, #12, 1991.

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