english**MiniSpacer® Dual Spray Metered Dose Inhaler Adapter**

With Counter Incrementing Actuator
Bi-Directional Spray and Cap
15 mm O.D. / 15 mm I.D. ISO Connections

INDICATIONS FOR USE

Model number 1543A is a single patient, disposable device for dispensing pressurized metered dose inhaler (pMDI) medication into a breathing circuit, as prescribed by a physician or other licensed health care practitioner. The device is indicated for patients on a breathing circuit, for whom aerosol medication has been prescribed, in short and long term critical care environments. REF 1543A is intended for use only when connected to a 15 mm fitting.

DIRECTIONS FOR USE**Installing the MINISPACER Adapter in the Patient Circuit**

1. Remove the MINISPACER adapter from package.
2. Install the MINISPACER device in the patient circuit as indicated here or according to facility guidelines:
 - For a 22 mm circuit, install the MINISPACER between the patient wye and endotracheal tube.
 - For a 15 mm circuit, install the MINISPACER between the patient wye and the inspiratory limb of the patient circuit.

CAUTION : Ensure the device is securely installed before use.

NOTE : When correctly installed, the MINISPACER MDI port will be vertical and the pMDI canister will point downward during treatment.

3. Place the cap firmly on the MINISPACER port.

Administering MDI medication via the MINISPACER Adapter

1. Uncap the MINISPACER MDI port.
2. Insert the MDI canister stem into the MINISPACER port.
- NOTE : If canister stem features a dose-counting mechanism in the tip, align the MINISPACER's gear with the dose-counter. When correctly aligned, the canister's number dial will be visible through the MINISPACER window.
3. Coordinate MDI actuation with the beginning of inspiration.
- CAUTION : Support the MINISPACER adapter while depressing the canister.
4. Administer the dose as ordered.
5. After medication has been delivered as prescribed, remove the pMDI canister and firmly cap the MINISPACER port.
6. Monitor the patient response.

NOTE : Discard the MINISPACER adapter when replacing the circuit according to facility guidelines, not to exceed two weeks.

WARNING : Do not clean and reuse or sterilize. Potential risks associated with cleaning and reusing, reprocessing and/or sterilization include: cross-contamination or infection resulting from inadequate cleaning, product degradation, and/or failure of the device to perform as intended.

Symbol Glossary

	Manufacturer		Consult instructions for use
	Use by Date		Authorized representative in the European Community
	Batch code/lot number	Rx Only	Prescription only. U.S. Federal law restricts this device to sale by or on the order of a physician or properly licensed practitioner.
	CE Mark		Do Not Reuse
	Catalog, reorder or reference number		Caution. The symbol can also be used to mean Attention, see Instructions for Use.

Item No.	Description
REF 1543A	MiniSpacer® Dual Spray MDI Adapter, 15 mm O.D. / I.D.

Thayer Medical Corporation
4575 S. Palo Verde Rd.
Tucson, AZ 85714-1961 USA
www.thayermedical.com
Made in USA

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